



ACD-BL Series Micro control with printer &USB

| Dental Autoclave

☰ Details Info

Specification:

Model	ACD-12BL	ACD-18BL	ACD-23BL
Description	Small Autoclave	Small Autoclave	Small Autoclave
Class	B	B	B
Standard	BS EN 13060:2004 + A2:2010		
Frequency	50Hz	50Hz	50Hz
Plug	3 hole plug	3 hole plug	3 hole plug
Wattage	2.4 KW	2.4 KW	2.4 KW

Sterilization temperature	105 - 134°C	105 - 134°C	105 - 134°C
Max working pressure	0.21-0.23MPA	0.21-0.23MPA	0.21-0.23MPA
Diameter of chamber	20 CM	25 CM	25 CM
Depth of chamber	36 CM	35 CM	45 CM
Volume of chamber	12 L	18 L	23 L
Material of chamber	Stainless steel 304	Stainless steel 304	Stainless steel 304
Colors available	White	White	White
Screen type	LCD	LCD	LCD
Screen size	6 X 4.25 cm	6 X 4.25 cm	6 X 4.25 cm
Language available	English, Italian, German		
Lock system of door	Mechanic + electrical lockers		
User password	Yes	Yes	Yes
Output	Thermo printer & USB	Thermo printer & USB	Thermo printer & USB
Net weight	51 kgs	53 kgs	55 kgs
Gross weight / carton	59 kgs	61 kgs	63 kgs
Product size	61 X 50 X 43cm	61 X 50 X 43cm	65 X 50 X 43cm
Package size	73 X 57 X 51cm	73 X 57 X 51cm	77 X 57 X 51cm
Min size of table for installation	54 x 41.5 cm	54 x 41.5 cm	54 x 41.5 cm

Function:

Fully Automatic

8 Preset sterilization programs

Ster. Temp.: 105-134°C

Ster. Time: 4-60 min

Dry time: 1-25min


Vacuum Times: 1-10 times

1. B&D / Helix / Vacuum testing program
2. Cleaning Program
3. Printer & USB Output
4. Big Stainless Steel Water Tanks
5. High quality bacteria filter

User Password

Program	121°C Solid	121°C Hollow	121°C Porous	121°C Solid	121°C Hollow	121°C Porous
---------	----------------	-----------------	-----------------	----------------	-----------------	-----------------

Vacuum Times	1	3	3	1	3	3
Temp (°C)	121	121	121	134	134	134
Pressure(KPA)	110	110	110	210	210	210
Time(Min.)	20	20	20	4	4	4
Program	Cotton	Prion	User defined	BD/Helix Test	Vacuum Test	Clean
Vacuum Times	3	3	User defined	3	1	3
Temp (°C)	134	134	User defined	134	/	105
Pressure(KPA)	210	210	User defined	210	-80	20
Time(Min.)	4	19	User defined	3.5	15	5


 Head Office :1600 AVENUE DE LORIMIER BUREAU, 384 MONTRÉAL (QUÉBEC) CANADA H2K3W8 USAOffice:128 W SUFFLOK AVE,CENTRAL ISLIP,11722 N Middle east & Africa Office : warehouse 9 /shed 9, Technology park P.O. Box 54806 RAK FZE , UAE


 Email Us At:

sales@acculabusa.com middleeast@acculabusa.com


 Phone Support:

© Copyright 2014 by ACCULAB_USA - All rights reserved